

WHEN & WHERE TO USE



HARD, NON-POROUS SURFACES

(stainless steel, chrome, glass, plastic, vinyl, etc.)
Formulated to clean and disinfect hard, non-porous surfaces.



HIGH-TOUCH SURFACES

(doorknobs, bedrails, light switches, handrails, call buttons, telephones, plumbing fixtures, etc.)
Clean and disinfect surfaces with frequent hand contact in patient care areas daily, or when visibly soiled – especially when the patient is in Contact Precautions.



PATIENT CONTACT SURFACES

(stretcher, exam table, procedure table, chair, commode chair, counter, patient bed, etc.)
Clean and disinfect patient contact surfaces regularly – and whenever surfaces are contaminated with blood or other bodily fluid.



EQUIPMENT & DEVICES

(monitoring equipment, diagnostic equipment, digital monitors, mobile devices, mobile medical equipment and other commonly used equipment)
Clean outside surfaces of commonly used medical equipment and devices after each patient use.

HOW TO USE



1. PULL WIPE USING ARROW GUIDES

Pull a cloth wipe from the container using the two arrow guides on the lid, removing the cloth at a 90° angle for easy separation along the perforation.



2. WIPE SURFACE UNTIL WET

Unfold the cloth and wipe surfaces until visibly wet. Ensure at least 5 minutes of contact time.



3. LET AIR-DRY FOR 5 MINUTES

Do not wipe surfaces dry; let air-dry to ensure full antimicrobial efficacy.



4. WIPE RESIDUE, IF NECESSARY

After 5-minute contact time, wipe or rinse away any residual from shiny surfaces, if necessary.

FOR GROSSLY SOILED OR CONTAMINATED SURFACES
TWO-STEP CLEAN + DISINFECT

1. CLEAN

Wipe surface and remove soils and bioburden. Always follow your organization's PPE requirements when cleaning up bodily fluid.

2. DISINFECT

Use a second wipe to wipe surface until wet. Allow to air dry for at least 5 min.



SURFACE WIPES

Cloth wipes are non irritating, but they are designed for cleaning surfaces - not for use on skin



DO NOT RE-USE

Wipes cannot be reused.



DISCARD IN TRASH

Wipes are not flushable. Discard them in trash after single use.



KEEP CONTAINER CLOSED

Keep center cap on wipe container closed to prevent moisture loss that could impact wipe efficacy.

A SMARTER, SAFER WAY TO CLEAN + DISINFECT

Citric acid based products deliver powerful efficacy against pathogens - without the harmful, irritating effects of other chemical disinfectants - in an easy-to-use wipe format that enables one step, no-rinse cleaning and disinfection.